



For Cat. Nos.SL-100, SL-150, SL-200

LEAKTIGHT SHARPS DISPOSAL CONTAINER

Post Medical's Leak-Tight sharps disposal containers are enclosed. These containers may be used as freestanding units or can fit into most travel bags. These containers comply with or exceed OSHA regulations for sharps disposal containers.

WHEN TRANSPORTING PRIOR TO FINAL DISPOSAL

1. Snap tethered cap securely over top iris of container until it clicks closed
2. Unsnap tethered cap once ready for reuse prior to final disposal

WHEN CONTAINER IS READY FOR FINAL DISPOSAL:

1. Follow step 1 above
2. Dispose the filled container according to all state or hospital requirements

A significant feature has been added to your Post Medical Sharps Container—after the container is filled, the self-contained locking screw cap is securely screwed onto the top making the unit **leak-tight**

This practical modification benefits everyone handling these containers--from environmental services through nursing personnel. The danger associated with spillage/leakage from overturned sharps collection containers is virtually eliminated. Along with the more secure screw cap closure, please note the container's unitized construction.