

For Cat. Nos. WD-110, WD-150, WD-200

LEAKTIGHT SHARPS DISPOSAL CONTAINER

Post Medical's Leak-Tight sharps disposal containers are enclosed. These containers may be used as freestanding units or can fit into most travel bags. These containers comply with or exceed OSHA regulations for sharps disposal containers.

WHEN TRANSPORTING PRIOR TO FINAL DISPOSAL

1. Remove cap, located on side of unit
2. Screw cap on one to two turns making sure not to tighten over locking teeth on bottle
3. Unscrew and return to side of container after transport

WHEN CONTAINER IS READY FOR FINAL DISPOSAL:

1. Remove cap, located on side of unit
2. Screw cap securely onto top opening to lock unit
3. Dispose the filled container according to all state or hospital requirements

A significant feature has been added to your Post Medical Sharps Container—after the container is filled, the self-contained locking screw cap is securely screwed onto the top making the unit **leak-tight**

This practical modification benefits everyone handling these containers—from environmental services through nursing personnel. The danger associated with spillage/leakage from overturned sharps collection containers is virtually eliminated. Along with the more secure screw cap closure, please note the container's unitized construction.