

POST MEDICAL, INC.

LEAK-TIGHT* SHARPS DISPOSAL CONTAINER

Post Medical's Leak-Tight* sharps disposal containers are enclosed. These containers may be used as freestanding units or can fit into your existing metal cabinets located in patient rooms. This container complies with or exceeds OSHA regulations for sharps disposal containers.

WHEN CONTAINER IS READY FOR DISPOSAL:

1. If used in a metal cabinet, first, open metal cabinet.
2. Unwind cap, located on side of unit
3. Screw cap securely onto top opening.
4. Replace filled unit with new plastic insert.
5. Close and re-lock the metal cabinet.
6. Dispose the filled container according to hospital regulations.

**A significant safety feature has been added to your Post Medical Sharps Container -- after the container is filled, the self-contained locking screw cap is securely screwed onto the top making the unit leak-tight/*

This practical modification benefits everyone handling these containers -- from environmental services through nursing personnel. The danger associated with spillage/leakage from overturned sharps collection containers is virtually eliminated. Along with the more secure screw cap closure, please note the container's unitized construction -- the strongest most secure available.